SCHULER

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Ag/AgCl Electrode

Use:

The electrode metal used for the EEG electrode is a sintered Ag/AgCl disk with a diameter of 5mm or 10mm, inserted in a plastic body. Before first use or after a long period of non-use, place the device (not the cable) in 0.9% Sodium Chloride solution for a few minutes. To reduce skin resistance, it is necessary to clean and roughen the skin before applying the electrode. The electrode can be applied using an adhesive ring or adhesive paste. Suitable for medical devices for non-invasive applications that meet the respective applicable regulatory safety requirements.

Disinfection:

The electrode and cable must be disinfected before being used for the first time and after each use. Please only wipe disinfect, never immerse in liquid!

Cleaning:

Paste residues and other contaminants must be removed immediately after use. To do this, use a dry cloth and / or a cotton swab. Paste residues should not dry on the electrode. Contact with the sintered surface with metallic objects must be avoided. Do not clean them in an ultrasonic bath! Electrodes should be stored in a dry and dark place when not in use.

Useful service life:

With proper and professional use, cleaning, disinfection and storage, a service life of up to two years can be achieved.

Warning notice:

Before disinfecting and / or cleaning, disconnect the cable / electrode from the device or system. Disinfectants and instrument cleaning agents should only be used in the concentrations specified by the manufacturer. Please dispose of and replace damaged electrode leads (e.g. inner conductor is visible) immediately. The electrodes are intended for use on intact skin and are to be applied by medical professionals. The use of disposable electrodes is recommended if an increased risk of infection is identified in individual patients by the treating medical staff. Claims for compensation that are based on improper handling or use of the cable for other purposes shall not be accepted. All serious incidents that occur in connection with the electrodes must be reported to the manufacturer and the responsible authority

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